

33.5 cm

1. Line for water:

Pipe finished with 1/2", 2 cm above the floor
Connection to manual ball valve SA 1/2".


2. Line for compressed air

Pipe finished with 1/2", 2 cm above the floor
Connection to manual ball valve SA 1/2".

12.5 cm

Tubing

Tubing

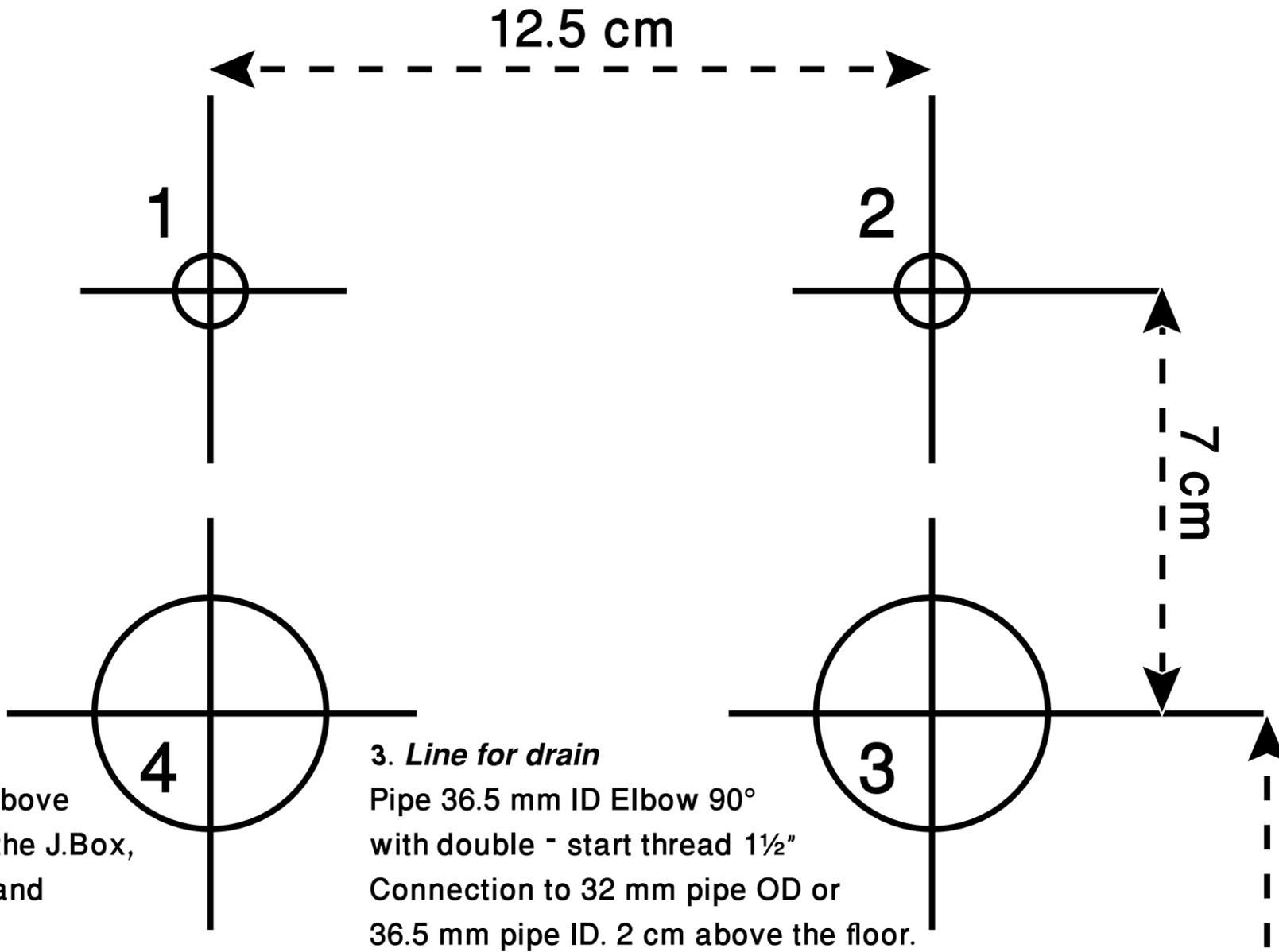

4. Line for vacuum

Pipe 36.5 mm ID, 2 cm above the floor, line begins at the J.Box, ends at separator tank and suction engine.

3. Line for drain

Pipe 36.5 mm ID Elbow 90° with double - start thread 1 1/2" Connection to 32 mm pipe OD or 36.5 mm pipe ID. 2 cm above the floor.

34.7 cm


Electric cabin

5. Main power line

230V, cable 3 X 1.5

7. Command line for door buzzer or an X-Ray,

If installed on the unit - cable 3 X 1.5

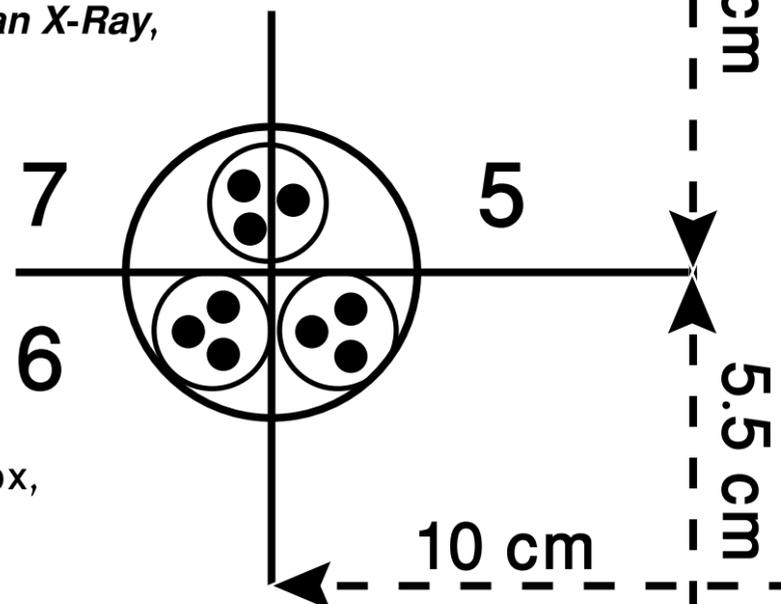

6. Electric command line for suction

Cable 3 X 1.5. Line begins at the J.Box, Ends suction engine.

10 cm

5.5 cm

11.7 cm

If located in front of chair base
The headrest should be

If located left to chair base
The headrest should be

External

J.Box