

13. CALIBRATION



CAUTION

During this operation there is X-ray output.

All the safety measures relevant to radioprotection must be observed.

If necessary, “ X-Mind® DC ” can be calibrated.

The relevant procedure is as follows:

a. Switch off the timer

b. Switch on the timer and keep the mA  button depressed

c. The display shows the “tube” message

d. Select X-RAY with the  button to calibrate

e. Take the exposure:

Press and keep the “RX”  button depressed until the beeping stops

stops the yellow  “X-RAY OUTPUT SIGNAL” LED goes off

f. Once the exposure has been taken, if the display does not show “ERRORS”, the calibration has been successfully completed, otherwise (*refer to §14 “ERROR MESSAGES”*)