

6. *Electric command line for suction Cable*
 3 X 1.5. Line begins at the J.Box, Ends suction engine.

7. *Command line for door buzzer or an X-Ray, If installed on the unit*
 cable 3 X 1.5


5. *Main power line*
 230V, cable 3 X 1.5

4. *Line for vacuum Pipe*
 36.5 mm ID, 2 cm above the floor, line begins at the J.Box, ends at separator tank and suction engine.

3. *Line for drain*
 Pipe 36.5 mm ID Elbow 90° with double - start thread 1½" Connection to 32 mm pipe OD or 36.5 mm pipe ID. 2 cm above the floor.






1. *Line for water:*
 Pipe finished with ½", 2 cm above the floor Connection to manual ball valve SA ½".

2. *Line for compressed air*
 Pipe finished with ½", 2 cm above the floor Connection to manual ball valve SA ½".

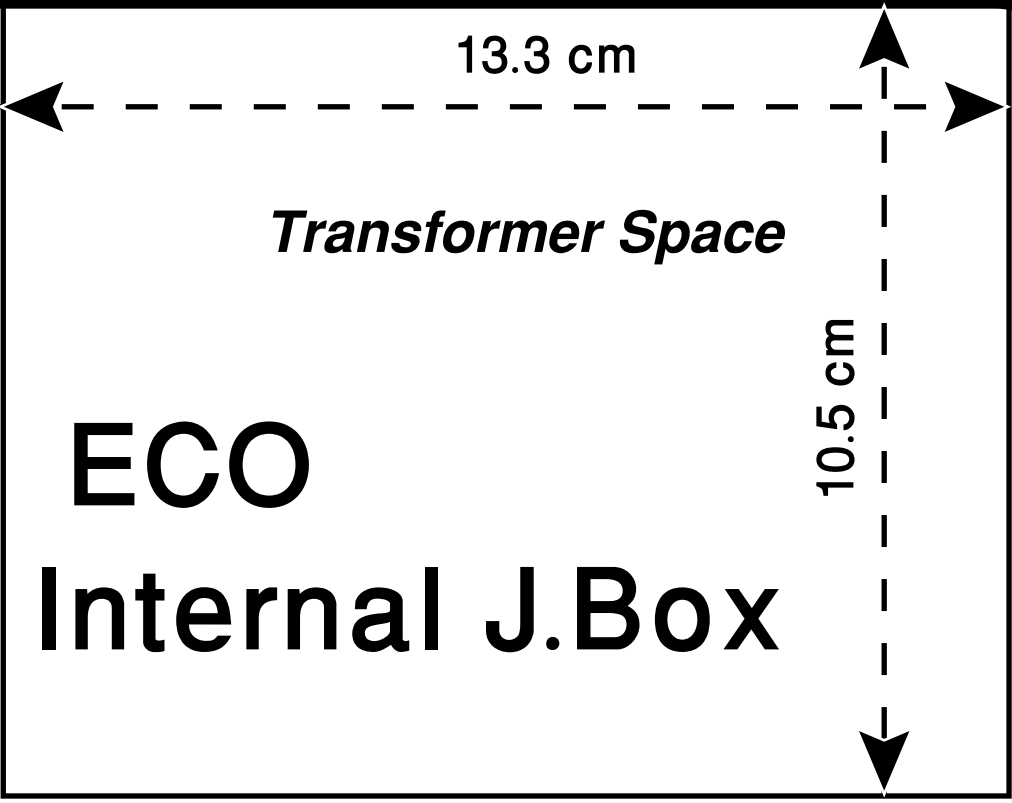

ECO
 Internal J.Box

1.2 cm

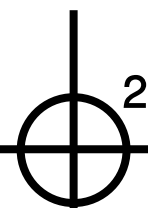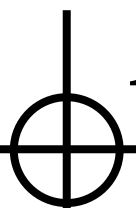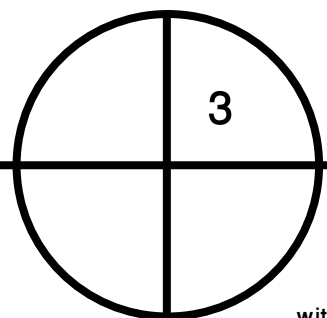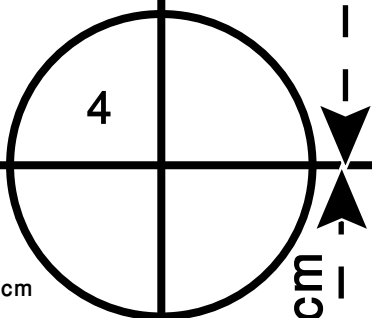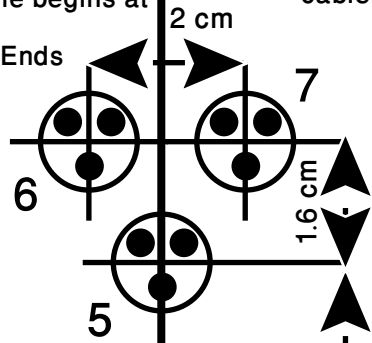

4.5 cm

15.5 cm

6.7 cm

7 cm

4 cm

8.3 cm